

LIFEPAK[®] 1000 DEFIBRILLATOR

Getting Started

Thank you for purchasing the LIFEPAK 1000 defibrillator. You have joined lifesaving teams around the world who are committed to meeting their diverse defibrillation needs. This guide provides the information you need to get started.



Please complete these five easy steps to get started.

1 Check the contents of the LIFEPAK 1000 defibrillator box.

- a. Carefully remove the defibrillator from the boxes you receive.
- b. Check the contents of each box to be sure everything listed on the packlist is included.
- c. Examine the outside of the defibrillator, battery and any accessories for signs of damage that may have occurred during shipping.
- d. If possible, save the shipping boxes and inserts in case you need to ship the defibrillator in the future.

2 Install the LIFEPAK 1000 battery in the defibrillator.

The LIFEPAK 1000 defibrillator can be powered by two types of batteries:

- a nonrechargeable Lithium manganese dioxide battery
- a rechargeable Lithium-ion battery

For rechargeable batteries only: You must fully charge the battery before first use. Insert the battery into the battery charger and verify that the green LED flashes to indicate that the battery is charging. Continue charging the battery until the LED changes to steady green. Only the LIFEPAK 1000 Defibrillator Battery Charger can be used to charge the battery.

For both battery types: Check the charge level before installing the battery in the defibrillator. Press the button on the battery and check the fuel gauge (row of green lights above the battery button). You should see four green lights illuminated.

Note: Always carry a spare, fully-charged battery.


To install the battery:

- a. Hold the battery so that the label faces away from you, and then insert the connector end of the battery into the battery compartment.
- b. Slide the battery all the way in until it latches securely. The defibrillator turns on and then off automatically and updates the readiness display.
- c. Verify that the OK symbol and the battery status symbols are visible in the readiness display.

3 Connect the therapy electrodes to the defibrillator.

QUIK-COMBO® pacing/defibrillation/ECG electrodes with the REDI-PAK™ preconnect system are provided. These electrodes are designed for preconnection to the defibrillator.

- a. Insert the electrode connector as shown in the following illustration:

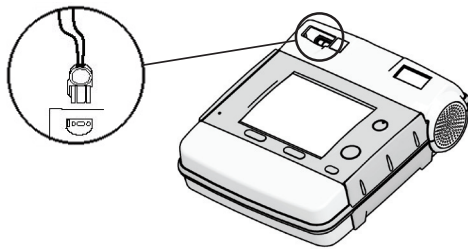

- b. Slide the electrode packet into the inner sleeve of the carrying case lid.

4 Familiarize yourself with the defibrillator and accessories.

Do not remove the defibrillator from its case.

A printed copy of the LIFEPAK 1000 Defibrillator Operating Instructions is included. For more information, see www.physio-control.com.

5 Customize the defibrillator operating settings if needed.

As determined by your medical director, some of the operating settings can be changed to match your emergency medical system protocol.

For further information please contact your local Physio-Control representative or visit our website at www.physio-control.com



Physio-Control Headquarters
11811 Willows Road NE
Redmond, WA 98052
Tel 425 867 4000
Fax 425 867 4121
www.physio-control.com